The Main Principle:

- ZD-LSBC gets low level laser radiation of 650nm wavelength, which can be absorbed by blood through nasal mucous membrane.
- Low level laser of 650nm wavelength is on the top of the HB spectral analysis , which can be absorbed strongly by HB.
- Low level laser of 650nm wavelength can make the extra fat wrapping around the RBC drop down. So the electriferous power of RBC recovers and the transmutation of RBC improves. Cardiovascular and encephalovascular diseases of lacking blood can be prevented and treated.
- With the dropping down of the exact fat, the combinative Oxygen of RBC will be increase deeply. The supply of Oxygen in body is improved at the same time.
- Low level laser of 650nm wavelength can correct the abnormal metabolism of fat, keeping the balance of fat level and the proportion of cholesterol and lecithoid.
- It can active the sympathetic nerve and the vice-sympathetic nerve, so the circulation of blood in the brain and all over the body will be improved obviously.

The functional structure and composition of product:



1. Coupling wire of laser

- 2. Socket of power wire
- 3. Indicating light of power
- 4. "+" of the power button
- 5. Button of power
- 6. " "of the power button
- 7. Power switch
- 8. Memory switch
- 9. Impulse switch
- 10. Laser switch
- 11. Laser inserter head
- 12. Nose clamp

Preparation before use :