



华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

北京金华汉新技术有限责任公司

BEIJING GOLDEN HUAHAN NEW TECHNOLOGY CO.,LTD.



# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

华汉针神

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

- **Standard Configurations**





华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

## • Chapter-1: Preparation Before Use



**Battery 3\*AA**



**AC/DC Power**

### a. Power ON/OFF:

Press the membrane switch button, the power LED is on in green light with a sound signal, Once again, power off.








# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

华汉针神

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

## • Chapter-1: Preparation Before Use

### b. Time setting

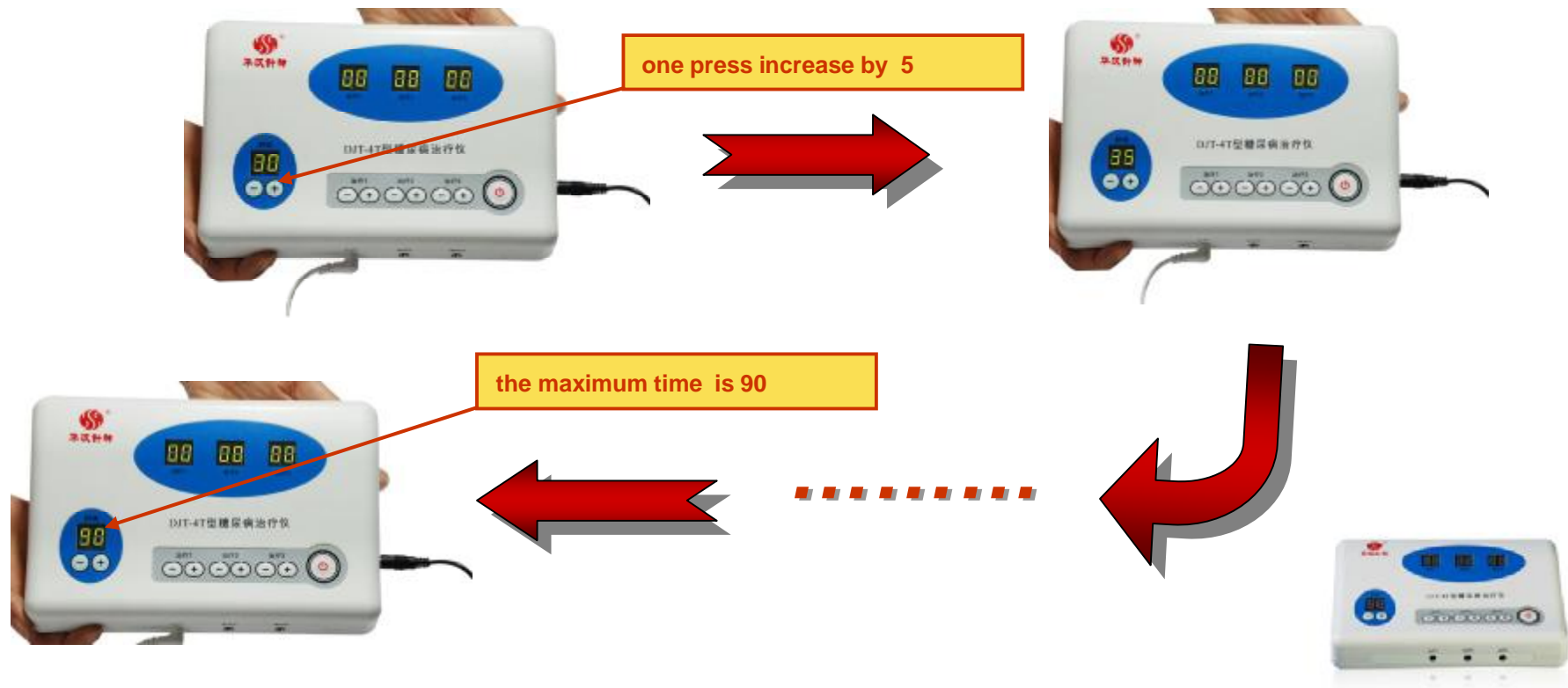




华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

## • Chapter-1: Preparation Before Use

### c. Therapy Intensity

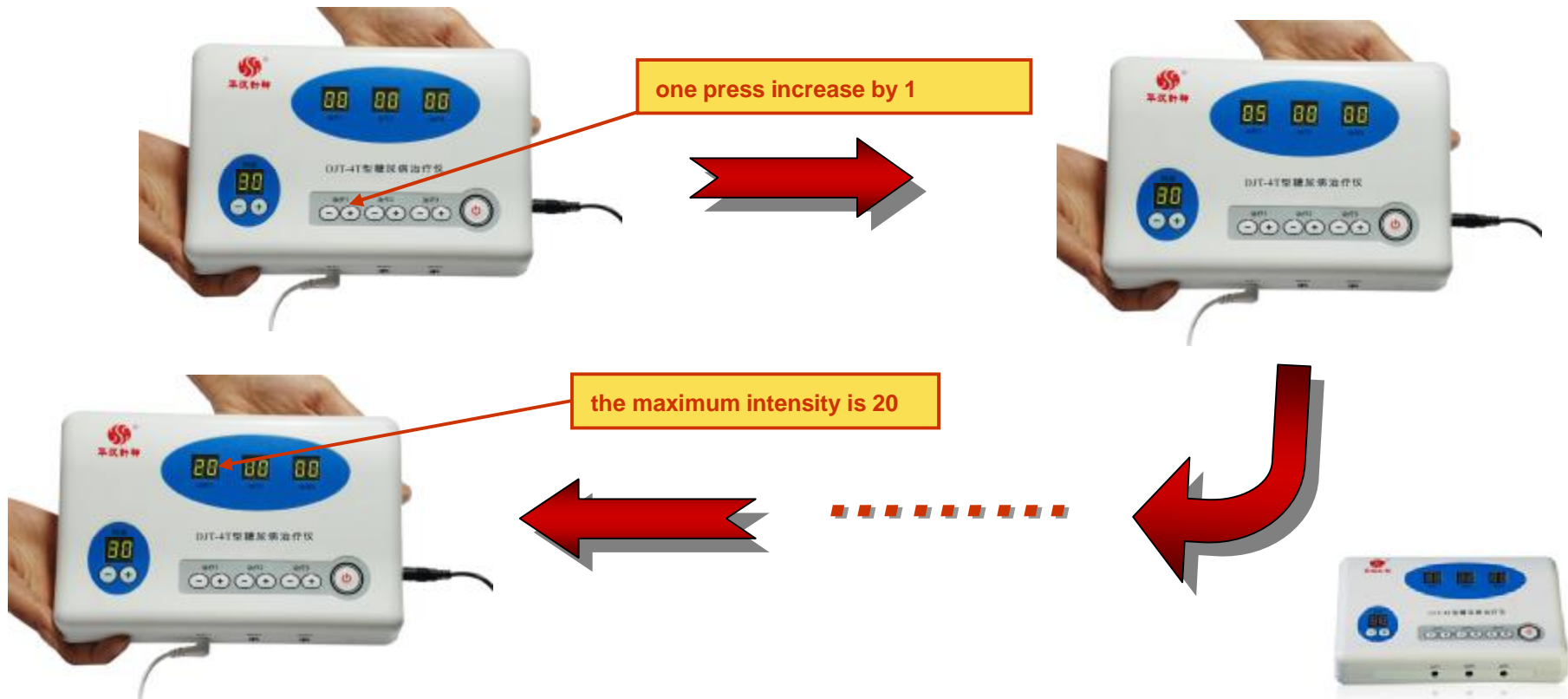



华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

## • Chapter-1: Preparation Before Use

When use 3\*AA batteries, pay attention to the positive “+” and negative “-” electrode.

When use power supply, be sure to connect AC/DC adapter first.



**Check the device and accessories carefully before usage and make sure all parts are in good conditions.**

Switch on the power, both the time area will show 30 and the output intensity will all show 00 with a sound signal, it shows the device works well, switch off the power and operate as follows.







华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

## • Chapter-2: Treatment Modes

### Treatment Modes Selection

- a).Choose therapy modes:** The device has five therapy ways as, a. Ear acupoints holographic therapy;  
b. Hand acupoints holographic therapy; c. Foot acupoints holographic therapy; d. SPA holographic therapy;  
e. Body acupoints therapy.

You'd better choose the prior therapy modes to treat diseases in the sequence from "a" to "e".

Remember to use treatment modes by turns.



- b).Treatment time:** Twice or three times per day, and 30 -45minutes each time. If you are available, more times and longer time is preferred. And You'll get excellent treatment results if you keep using it.





华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

## *Ear Acupoints Holographic Therapy*








华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

- *Ear Acupoints Holographic Therapy*

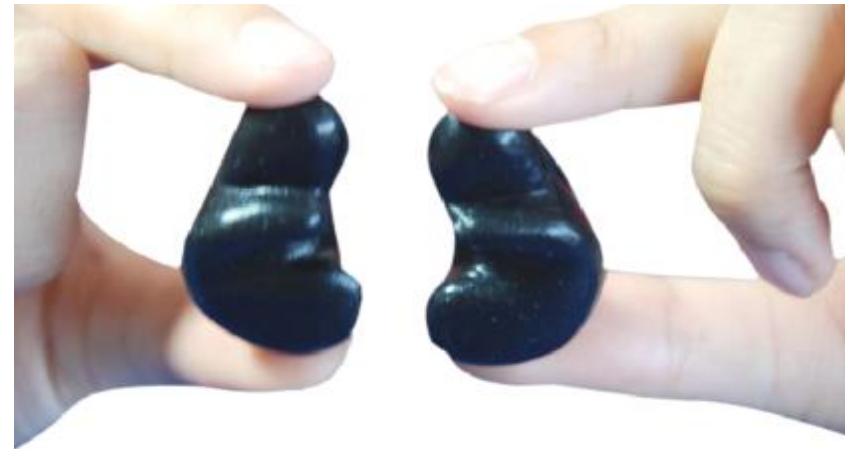

Left

Right

Before connecting to power supply, plug the two output electrodes respectively into two auricular moulds, choose the auricular moulds with right size of your external ears and make sure they are fit your ear and connected with ear skin well.






华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

- *Ear Acupoints Holographic Therapy*

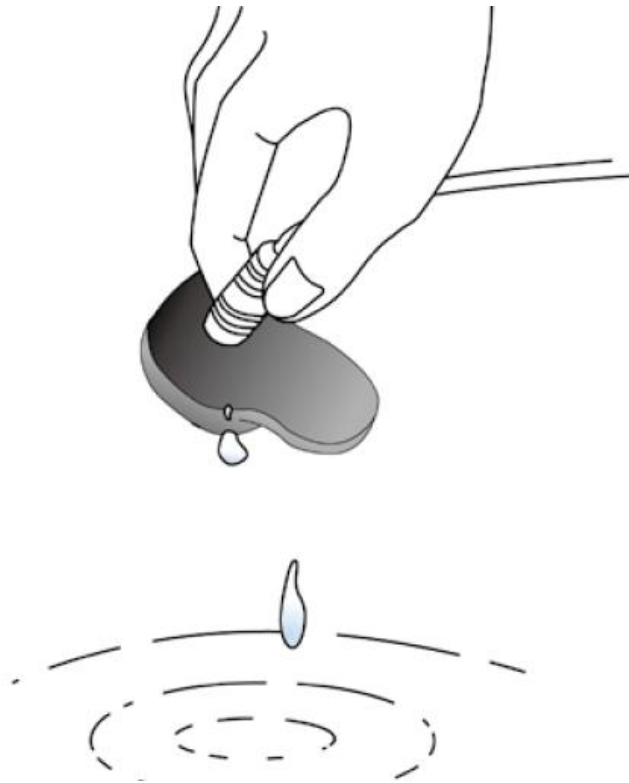

Before therapy, clean the ears' surface with water and make auricular moulds moist with water. This will make the stimulation evenly and gently avoiding ear pains.

Alternative use of wet and dry ear moulds, better results obtained.





华汉针神


# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT



- *Ear Acupoints Holographic Therapy*



1

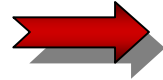


2

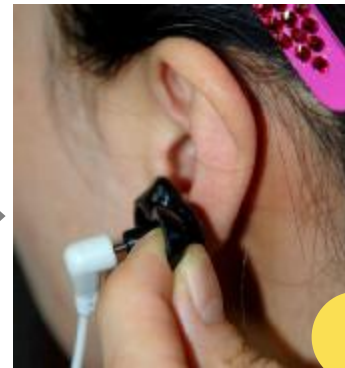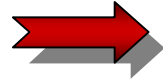


3



4

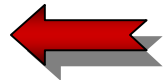

5

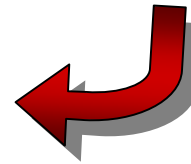

**Five Steps!**







华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

- *Ear Acupoints Holographic Therapy*



If auricular moulds are not well connected to your ears, you can use the auricular clamp to fix them. Firstly insert the opening ends of the clamp into the pillars of wire plug, which are on the auricular moulds, then adjust the length of the clamp, at last put the auricular clamp on your head to fix auricular moulds.







华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

- *Ear Acupoints Holographic Therapy*

**Remember to increase  
the intensity slowly !**



Switch on the device, both the power LED and therapy LED are lighted. Increase the output intensity until you feel your ears having acmesthesia, which shall be endurable and comfortable. It is normal that the feelings of two ears are different. After you have accustomed to the stimulation strength, keep it for 2 to 3 minutes, then increase the intensity gradually until the nerves and muscles around the eyes and the face have a kind of vibrant sense. By selecting this intensity, you can expect a better treatment result. 20-30 minutes later, switch off the device.





华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

## *Hand Acupoints Holographic Therapy*



北京金华汉新技术有限责任公司  
Beijing Golden Huahan New Technology Co.,Ltd.



华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

- *Hand Acupoints Holographic Therapy*



Before connecting to power supply, plug the two output electrodes respectively into two hand palm electrodes.





华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

- *Hand Acupoints Holographic Therapy*

**Remember to increase  
the intensity slowly !**



Switch on the device, both the power LED and therapy LED are lighted. Increase the output intensity until you feel your hands having acmesthesia, which shall be endurable and comfortable. It's normal that the feelings of two hands are different.

30-45 minutes later, switch off the device.







华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

## *Foot Acupoints Holographic Therapy*



北京金华汉新技术有限责任公司  
Beijing Golden Huahan New Technology Co.,Ltd.



华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

- *Foot Acupoints Holographic Therapy*



Before connecting to power supply, plug the two output electrodes respectively into two insole electrodes.





华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

- *Foot Acupoints Holographic Therapy*



**Put your feet on  
the front side!**

It will be more effective to treat  
after shower or foot bathing  
with feet moist with water





华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

- *Foot Acupoints Holographic Therapy*

**Remember to increase  
the intensity slowly !**



Switch on the device, both the power LED and therapy LED are lighted. Increase the output intensity until you feel your feet having acmesthesia, which shall be endurable and comfortable. It's normal that the feelings of two feet are different.

30-45 minutes later, switch off the device.







华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

## *SPA Holographic Therapy*



北京金华汉新技术有限责任公司  
Beijing Golden Huahan New Technology Co.,Ltd.




华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

- **SPA Holographic Therapy**

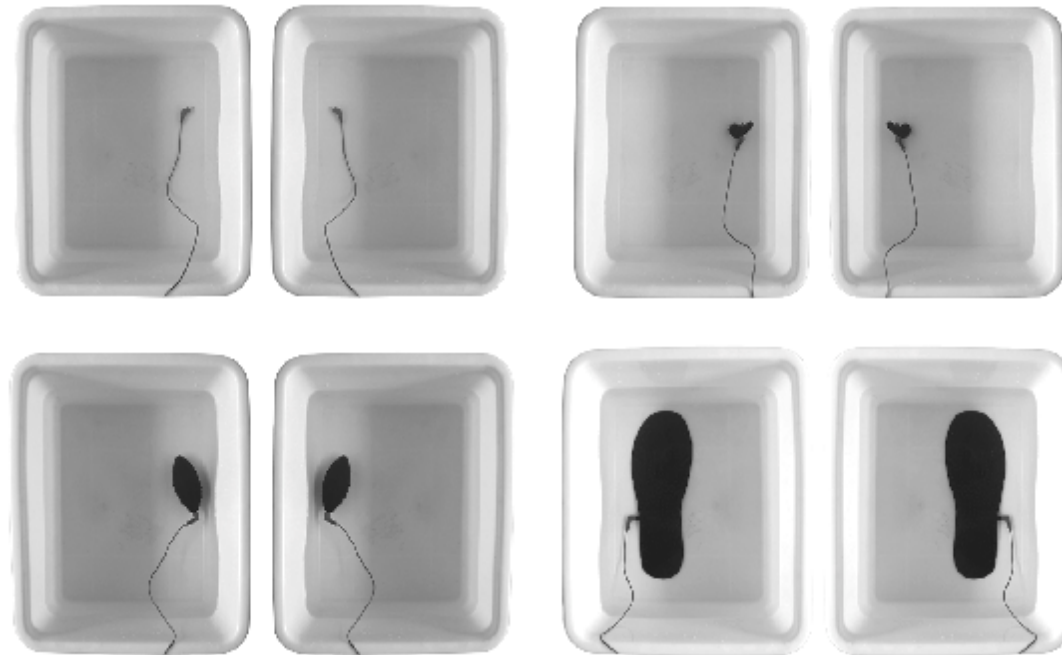

Connect two auricular moulds ( or either hand palm electrodes or insole electrodes) with two output electrodes, then put these two separately into two basins with water, then put your hands or feet into the water basins, follow the method of ear acupoint holographic therapy.





华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

- **SPA Holographic Therapy**

**Remember to increase  
the intensity slowly !**



Switch on the device, both the power LED and therapy LED are lighted. Increase the output intensity until you feel your hands or feet having acmesthesia, which shall be endurable and comfortable. It is normal that the feelings of two hands or feet are different.

30-45 minutes later, switch off the device after the treatment.







华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

## *Body Acupoints Therapy*






华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

- **Body Acupoints Therapy**

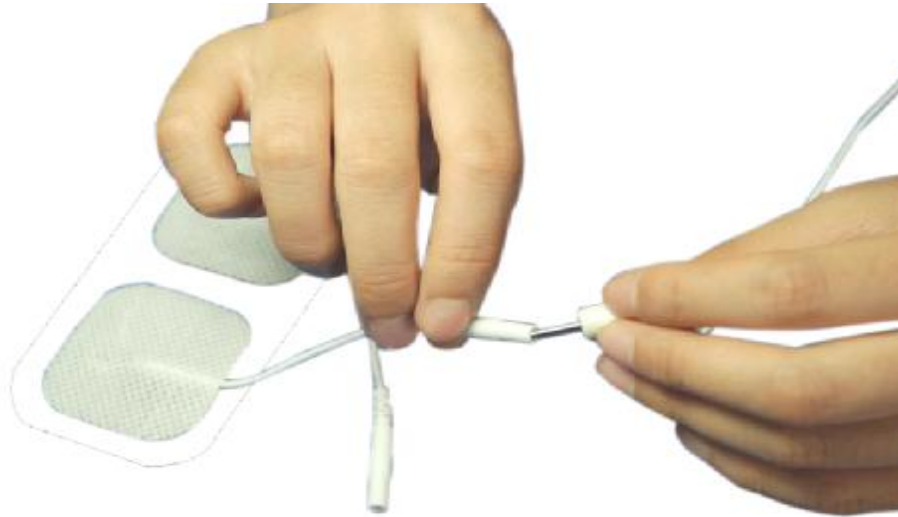

Before connecting to power supply, plug the two output electrodes respectively into two self-adhesive electrode pads.

Remove the transparent plastic films from the surface of electrode pads, put two pads on Ashi points or selected acupoints, then follow the method of ear acupoint holographic therapy.





华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

## • *Body Acupoints Therapy*

**Remember to choose acupoints tailed to your symptoms !**

### *Polydipsia:*

- (53)Feishu-(53)Feishu,
- (63)Pishu-(63)Pishu,
- (60)Yishu-(60)Yishu,
- (66)Shenshu-(66)Shenshu,
- (108)Zusanli-(115)Sanyinjiao,
- (126)Taixii-(126)Taixi,
- plus (59)Geshu-(59)Geshu,(85)Yuji-(85)Yuji

### *Diuresis*

- (66)Shenshu-(66)Shenshu, (60)Yishu-(60)Yishu,
- (61)Ganshu-(61)Ganshu,
- (126)Taixi-(126)Taixi,
- (119) Taichong-(119) Taichong,
- (115)Sanyinjiao-(115)Sanyinjiao

### *Polyphagia:*

- (63)Pishu-(63)Pishu,
- (59)Geshu-(59)Geshu,
- (42)Guangyuan-(42)Guanyuan,
- (64)Weishu-(64)Weishu;







华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

- **Body Acupoints Therapy**

**Remember to increase  
the intensity slowly !**



Switch on the device, both the power LED and therapy LED are lighted. Increase the output intensity until you feel your body acupoints area having acmesthesia, which shall be endurable and comfortable. It is normal that the feelings of two body parts are different.

30-45 minutes later, switch off the device after the treatment.





华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

- **Remarks:**

- *After using, stick the transparent plastic films to the surface of electrode pads to keep them clean.*
- *Please consult your TCM physicians for acupoints selection. It can achieve significant treatment results by just using Ashi points' treatment method.*
- *Ashi point is a painful point or area on your body surface.*
- *The stickness of body electrode pads will decrease after using for a long period of time. Clean them by water but not dip into water, and make them air-dried and stick them with dustproof plastic film to prolong their life.*





华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

## • Chapter-4: Trouble Shooting

*If the device works abnormally, check and eliminate faults according to Tab.2 Troubles and solutions in the operation manual. When no solution is found, please contact manufacturer or local dealers directly. Dismantlement and repair without authorization are strictly prohibited.*

- a). When auricular moulds, hand palm electrodes, self-adhesive electrode pads, or electrode insoles become dirty after many times of use, they can be cleaned with water and then be air-dried. Do not clean them with alcohol; do not store them in metal container. If there is no curative effect, or the electrode pads becomes low viscosity, please replace a new one.
- b). If device surface becomes dirty, use cloth dipped with neutral clean liquid to wipe, do not use organic dilution or gasoline for cleaning purpose.
- c). Do not store the device in humid or high temperature with direct sunshine.
- d). Keep the device out of reach of children.







华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT

## • *User Tips:*

- *Treatment time: 30-45 minutes each time, 2-3 times daily. The longer time, the more times, the better results. If you are busy, one time a day is acceptable.*
- *Treatment selection: ear, hand, foot, SPA therapy at first, then body acupoint treatment. Different diseases, different ways of treatment, always choosing the nearest projecting parts to do treatment. For example: head diseases - ear therapy; upper limbs diseases - hand therapy, lower limbs diseases - foot therapy. If body treatment, select the acupoint which is nearest to the pathological part of the body. Acupoints in hand & face are not suggested to select. Never select acupoints on the spine and heart area of the body.*
- *Treatment wire: use it gently especially when you plug in or out, make sure you have held the root of the electrode.*
- *Clean the treatment part before therapy.*





华汉针神

# 华汉针神 DJT-4T 型糖尿病治疗仪使用讲解

DJT-4T OPERATION INSTRUCTIONS OF DIABETES TREATMENT INSTRUMENT



THANK YOU



北京金华汉新技术有限责任公司  
Beijing Golden Huahan New Technology Co.,Ltd.